BOND-PRIME Cleaning Kit

Catalog No: CS9300

Leica Biosystems Newcastle Ltd
Balliol Business Park West
Benton Lane
Newcastle Upon Tyne NE12 8EW
United Kingdom
☎ +44 191 215 4242

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Catalog No: CS9300

Intended Use

The intended use of the BOND-PRIME Cleaning Kit is to periodically clean the BOND-PRIME ARC (Active Reagent Control) Array consisting of 24 ARC Modules (module used to hold a slide during the staining process).

Summary and Explanation

The BOND-PRIME Cleaning Kit is a set of ready-to-use cleaning reagents provided in a reagent tray. Routine use of the BOND-PRIME Cleaning Kit reduces the chromogenic residue on the ARC Modules.

Reagents Provided

Reagents sufficient for 20 cleans:

1. Cleaning Component A (30 mL) proprietary cleaning reagent
2. Cleaning Component B (30 mL) <5% sulphuric acid
3. Cleaning Component C (30 mL) proprietary cleaning reagent

Dilution and Mixing

The BOND-PRIME Cleaning Kit is optimized for use on the BOND-PRIME automated system. No manual reconstitution, mixing, dilution, or titration is required. BOND-PRIME mixes components to the correct concentrations on the instrument when cleaning is initiated.

Materials Required but Not Provided

No additional materials are required to use the BOND-PRIME Cleaning Kit.

Storage and Stability

Store at 2–8 °C. The product is stable under these conditions up to the expiry date indicated on the container and tray handle labels.

There are no obvious signs to indicate instability of this product. Return to 2–8 °C immediately after use.

Storage conditions other than those specified above must be verified by the user.¹

Precautions

- If a spill occurs on-board the BOND-PRIME automated system, it should be cleaned up as soon as possible using an appropriate absorbent material.
- To obtain a copy of the Material Safety Data Sheet contact your local distributor or Leica Biosystems' regional office, or visit the Leica Biosystems' web site: www.LeicaBiosystems.com
- Specimens, before and after fixation, and all materials exposed to them, should be handled as if capable of transmitting infection and disposed of with proper precautions.² Never pipette reagents by mouth and avoid contacting the skin and mucous membranes with reagents or specimens. If reagents or specimens come in contact with sensitive areas, wash with copious amounts of water. Seek medical advice.
- Consult Federal, state and local regulations for disposal of any potentially toxic components.

Instructions for Use

The BOND-PRIME Cleaning Kit was developed for use on the BOND-PRIME automated system using an installed protocol. The user loads the BOND-PRIME Cleaning Kit onto the BOND-PRIME as instructed by the BOND-PRIME User Manual. The BOND-PRIME automated system schedules and executes the cleaning protocol that returns the ARC Modules to a state suitable for staining.

Further information can be found in the Cleaning and Maintenance Section of the BOND-PRIME User Manual.

Troubleshooting

Refer to the BOND-PRIME User Manual.

If unexpected staining is observed that cannot be explained by variations in laboratory procedures, and a problem with the cleaning system is suspected, contact your local distributor or the regional office of Leica Biosystems immediately.

Further Information

Further information on immunostaining with BOND-PRIME reagents can be found in the BOND-PRIME User Manual.

References

1. Clinical Laboratory Improvement Amendments of 1988, Final Rule 57 FR 7163 February 28, 1992.
2. Villanova PA. National Committee for Clinical Laboratory Standards (NCCLS). Protection of laboratory workers from infectious diseases transmitted by blood and tissue; proposed guideline. 1991; 7(9). Order code M29-P.

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Leica Biosystems Inc
Pty Ltd
LBS Deutschland GmbH